lower arm may significantly shorten the life of your orthosis (e.g. racquet/bat/club sports, aggressive cycling, weights)

Consult your physician if planning any long flight (>3 hours)

Loosening & tightening straps

You may adjust strap nearest to the elbow to maximise comfort and support.

You may remove the microhook easy-fix tabs under the overlap (if not already done by clinician) to make adjustment easier.

The compliance strap, near the wrist, may have been locked by the physician to avoid unauthorised removal but can be loosened or tightened by a limited amount to maintain a snug fit.

If authorised by your physician, you may adjust the hook & loop tab across the back of the hand to maximise support and comfort.

Care & Comfort:

Maintain a snug fit. Follow the advice of your physician with regard to adjustment of the straps.

The orthosis is breathable and able to get wet. The Dermawick liner dries out relatively quickly but may remain damp for 2-4 hours. You may pat it dry with a towel or wrap it in a swim towel. You may further speed up the drying process by using a hair dryer on the COOL setting and large air-flow. Take care not to heat & soften the orthosis. If your physician allows you to loosen or remove the orthosis, then the drying process can be quicker. Keep your orthosis clean. Loosen slightly and rinse it with luke-warm water. Do not use soap directly on the orthosis unless it can be fully rinsed off as otherwise residue may cause skin irritation. Re-tighten the orthosis.

If the hook & loop closure strap on the back of the hand becomes worn and opens too easily, then remove the narrow strip to expose fresh hook.

See www.fastformmedical.com/patients/

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Patient Information

Care of your Short Arm Multifunctional Orthosis or Thumb Spica Multifunctional Orthosis

Warnings & Precautions:

Follow the advice of your physician regarding elevation of the injury and management of swelling.

Consult your physician if any unusual swelling, increasing discomfort, pain, numbness or discolouration occurs.

If you feel that your orthosis does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.

If your orthosis breaks or cracks, ask your healthcare professional if a replacement is required.

Wearing the orthosis too tight may result in skin irritation or discomfort.

The orthosis may soften and lose its desired shape when exposed to heat over 57°C (135°F). If exposed to strong sunlight, cover it with clothing or stockinet. Be aware that the product may soften if left in a car during very hot weather or under strong sunlight.

Do not apply household cleaners or personal hygiene products directly to the product as it may affect durability or irritate the skin.

Follow your physician's advice regarding exercise. The product is suitable for moderate exercise, but vigorous exercising of the

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Consultez www.fastformmedical.com/patients/

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FastForm Research Ltd.,
Breg, Inc.,