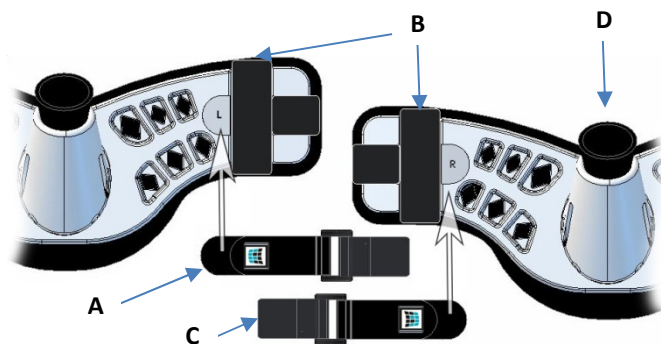
IFU-HTSO

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Hand-based Thumb Spica Orthosis

INSTRUCTIONS FOR USE

Before using the device, please read the following instructions completely and carefully. Correct application and care is vital to the proper functioning of the device

INTENDED USE / INDICATIONS:

- Treatment of ulnar collateral ligament injury or to provide support and immobilisation in the treatment of soft tissue injuries
- Immobilization and stabilization of the first MCP joint.

CONTRAINDICATIONS:

Simple, complex, compound and/or comminuted fractures. Not for treatment of an open wound.

WARNINGS AND PRECAUTIONS:

- Device to be fitted by a qualified healthcare professional.
- Patients should consult their medical professional immediately if they experience any pain, swelling, sensation changes, or any unusual reactions while using this device.
- This Device should not be used on a patient with polyester or elastane allergy.
- Ensure the orthosis is not too hot prior to application.
- Do not apply over open wound.
- Instruct the patient on correct adjustment of the strap to maintain snug fit.
- Remove the device before any re-molding with a heat-gun, to avoid burning the patient.
- The orthosis will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources (e.g. ovens, flames, sunny car windows, very humid conditions) for long periods.
- This device reduces mobility & dexterity. Do not operate heavy equipment.

1. SELECT & GAUGE

- Select orthosis size based on sizing chart on package.
- Remove the backing paper from the round end (A) of the strap.
 - To treat Right-hand, press the round end of strap firmly into the "R" recess.
 - To treat Left-hand, press the round end of strap firmly into the "L" recess.

2. HEAT

- **Fastform® or Nu-Wave IR Oven (1500W):** Lay the orthosis on the FastForm® rack, with the fabric down ensuring the plastic does not touch the sides. Heat the HTSO at **60% Power** (setting 6) for **4 mins** or per the specific settings on your appliance. Check that all areas are fully malleable. If not fully malleable, re-heat in 1-minute increments.
- **Convection oven:** Place the orthosis (fabric side down) into a standard convection oven at 75-85°C (167-185°F) for 9 minutes or until it becomes soft / malleable. DO NOT heat for a total time greater than one hour.
- The HTSO is not suitable for heating in a waterbath.

3. APPLY

- Place the HTSO over the thumb.
- Remove the 2nd backing paper (C) from the strap.
- Gently stretch the palmar wing across the palm and overlap on the back of the hand. Press the adhesive tab (C) into the soft PolytrexX®.
- Confirm good alignment and positioning of the device particularly at 5th metacarpal and the palmar crease.
- Roll back the thumb liner (D) to form a comfort edge.

4. MOLD

- Wrap an elastic bandage (not included) firmly but not tightly to conform the entire orthosis. A damp bandage accelerates cooling.
- Manipulate the device into the most appropriate position for healing.
- Rest patient's arm on a table until the orthosis is rigid then remove the elastic bandage.

5. ADJUSTMENT

- Once the orthosis is set, articulate both hinges (B) for easier removal.
Note: the physician should advise the patient on how to adjust the strap for a snug fit.
Note: After removal the orthosis can be reformed with a heat gun in a localised area or in an oven. See www.FastFormMedical.com for further details.

GETTING IT WET

If allowed by the physician, it is acceptable for the orthosis to become wet during use. Note: It may take a number of hours to become completely dry. If allowed by the physician to loosen or remove the orthosis, the patient may dry it using a hair dryer on a cool setting.

CLEANING INSTRUCTIONS:

To clean the orthosis, rinse it in warm water. Note: it is not recommended to use household cleaners or personal hygiene products directly on the orthosis.

STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

WARRANTY: FastForm Research Ltd covers defects in workmanship and materials for up to 12 weeks after this device is dispensed. Damage caused by normal wear & tear, abuse or alterations is not covered.

For further details on heating & application method, see:

www.FastFormmedical.com or www.breg.com/fastform/HeatingTimes

-Custom moulded for single patient use.

-Not made with natural rubber latex -Rx only

Notice: while every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this device

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FASTFORM FastForm Research Ltd.,
BREG Breg, Inc.,