

## Materials required:

<p>Nuwave Primo Oven; Model 20750 Qty: 1</p>		 <p>FastForm Configuration</p>  <p>For setup training video, please scan QR code</p> <p>Or visit: <a href="https://www.fastformmedical.com/clinicians/">https://www.fastformmedical.com/clinicians/</a></p>
<p>FastForm Activation Rack Primo; PS-9825 Qty:1</p>		
<p>5" Extender ring; Nuwave PN# PMP6. (Breg PN# 9808). Qty:1</p>		
<p>Activation settings label; PS-2628 Qty:1</p>		

**Procedure**



Figure 1:

Open box and remove the Primo oven from its packaging. Standard Assembly



Figure 2:

On the bottom part of the oven, remove the grill pan and discard.

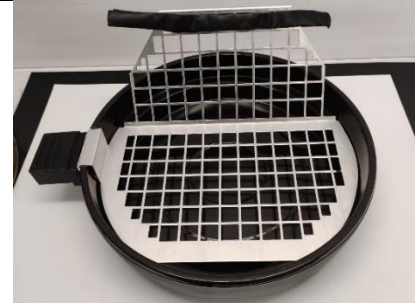


Figure 3:

Replace the grill pan with the FastForm Activation rack. Align the isolation socket with the aperture in the base. Ensure the rack is snug fitting.



Figure 4:

Mount the 5" extender ring onto the Assembly



Figure 5:

Mount the top assembly onto the bottom assembly.



Figure 6:

Insert the plug into the Isolation Socket (See note).



Figure 7:

Apply Activation Settings Label. Oven is now in its FastForm configuration & may be Powered up.

**Note:**

1. Isolation socket is for plug retention and power insulation only. There is no power transmitted to the FastForm Rack.
2. Failure to secure the plug into the isolation socket could expose the user to electrical risk under certain conditions. Check manual.