Patient Information HTSO

 English **Patient Information**

Care of your Hand-based Thumb Spica Orthosis

Warnings & Precautions:

- Follow the advice of your physician regarding elevation of the injury and management of swelling.
- Consult your physician if any unusual swelling, increasing discomfort, pain, numbness or discolouration occurs.
- If you feel that your orthosis does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.
- If your orthosis breaks or cracks, ask your healthcare professional if a replacement is required.
- Wearing the orthosis too tight may result in skin irritation or discomfort.

- The orthosis may soften and lose its desired shape when exposed to heat over 57°C (135°F). If exposed to strong sunlight, cover it with clothing or stockinet. Be aware that the product may soften if left in a car during very hot weather or under strong sunlight.
- Do not apply household cleaners or personal hygiene products directly to the product as it may affect durability or irritate the skin.
- Follow your physician's advice regarding exercise. The product is suitable for moderate exercise, but vigorous exercising of the lower arm may significantly shorten the life of your orthosis (e.g. racquet/bat/club sports, aggressive cycling, weights)

Loosening & tightening straps

- If authorised by your physician, you may adjust the hook & loop strap at the wrist to maximise support and comfort.

Care & Comfort:

1. Maintain a snug fit. Follow the advice of your physician with regard to adjustment of the straps.
2. The orthosis is breathable and able to get wet. You may pat it dry with a towel. You may further speed up the drying process by using a hair dryer on the COOL setting and large air-flow. Take care not to heat & soften the orthosis. If your physician allows you to loosen or remove the orthosis, then the drying process can be quicker.
3. Keep your orthosis clean. Loosen slightly and rinse it with luke-warm water. Do not use soap directly on the orthosis unless it can be fully rinsed off as otherwise residue may cause skin irritation. Re-tighten the orthosis.

If a serious incident occurs with this device, please report it to the Manufacturer and your Competent Authority.

See www.fastformmedical.com/patients/

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Consultez www.fastformmedical.com/patients/

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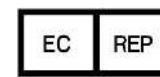
Visite: www.fastformmedical.com/patients/

82% PolytrexX Polymer/ Polymère/ Polimero
14% Polyester/ Polyester/ poliéster
3% Polyamide/ Polyamide/ Poliamida
1% Elastane/ élasthanne/ Elastano

Manufactured by/Fabriqu e par/Fabricado por:

FastForm Research Ltd.,

4 Lear Avenue, Airport Business Park, Killowen, Co. Waterford, IRELAND.

Tel: +353-51-306912

Email: info@FastFormResearch.com

Distributed in North America
Distribu e en Am rique du Nord
Distribuido en Norteam rica

Breg, Inc.,

2885 Loker Ave. East, Carlsbad, CA 92010, USA

Tel: 800-321-0607

www.breg.com